



SpeedySwab™

Rapid COVID-19 + FLU A&B Antigen Self-Test


SpeedySwab™ is an FDA EUA authorized Rapid COVID-19, influenza A and Influenza B Antigen Self-Test designed for nasal application and is highly effective in detecting all COVID-19 variants and sub-variants with a class leading COVID-19 Clinical Sensitivity of 92.5%.

The test uses the long-proven Lateral Flow Assay (LFA) technology. In three easy steps, simply swirl the soft tip of the nasal swab ½” – ¾” into each nostril for 15 seconds, insert the swab into a test tube with a buffer solution, and apply 4 drops onto the test card. Results will be available in 15 minutes.

Available with a 18-Month Shelf Life.

Positive Test Result

Flu A+B+ COVID-19

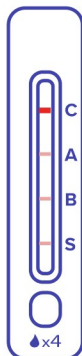


- C = Control Line
- A = Influenza A Line
- B = Influenza B Line
- S = COVID-19 (SARS-CoV 2)

Product Benefits

- **Reliable** design supporting highly sensitive detection of Influenza A and B, and COVID-19 genetic variants and sub-variants
- **Simple** format is designed for self-administration on adults, or by adults on children ages 2 and up for ease of use and provides results in 15 minutes
- Our **production capabilities** give us the ability to quickly produce large quantities of tests, allowing us to provide for the market as demand increases

Available Products

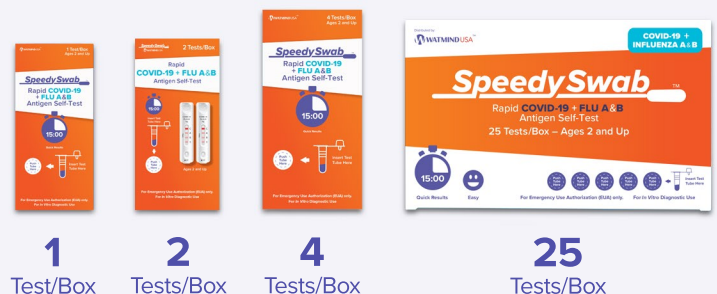

1 Test/Box

2 Tests/Box

4 Tests/Box

25 Tests/Box